

DISINFECTANT SOLUTION FOR MEDICAL DEVICES AND GENERAL SURFACES

ANIOSPRAY QUICK

DISINFECTANT SOLUTION

Fast disinfection of previously cleaned, non-immersible, non-invasive, alcohol-resistant surfaces, medical equipment and medical devices. Biocide: For general surfaces.

Ready to use solution

- Ethanol-based composition
- Broad activity spectrum in short contact time
- Does not leave any trace when drying
- Pleasantly fragranced
- Without dye



ANIOSPRAY QUICK

Disinfectant solution



Aniospray Quick
1L

Aniospray Quick
5L

PRODUCT CHARACTERISTICS

- Ready to use product
- Clear colourless solution
- Density: approx 0.9
- pH at +20°C: approx 5.5

MICROBIOLOGICAL PROPERTIES

STANDARDS	CONTACT TIME
EN 13697* (Bactericidal)	30 sec
EN 13697* (Yeasticidal)	30 sec
EN 13697* (Fungicidal)	5 min
EN 13727* (Bactericidal)	30 sec
EN 13624* (Yeasticidal)	30 sec
EN 13624 * (Fungicidal)	5 min
EN 14348* (Tuberculocidal)	30 sec
EN 14348* (Mycobactericidal)	5 min
EN 14476* (Effective against enveloped viruses)	30 sec
EN 14476* (limited spectrum virucidal) (Adenovirus / Norovirus (MNV))	5 min 30 sec/5 min
EN 16777* (Effective against enveloped viruses)	30 sec
EN 16777* (limited spectrum virucidal) (Adenovirus / Norovirus (MNV))	5 min 30 sec/5 min

* Clean conditions

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
ANIOSPRAY QUICK	12 x 1L	2084.073
ANIOSPRAY QUICK	4 x 5L	2084.034



CE
0459

LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

Distributed by:
LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

ECOLAB®

www.ecolab.com

LABORATOIRES
ANIOS
An Ecolab Company

www.anios.com

APPLICATION AREA

Fast disinfection of previously cleaned, non-immersible, non-invasive, alcohol-resistant surfaces, medical equipment and medical devices. Biocide: For general surfaces.

INSTRUCTIONS FOR USE

Ready-to-use. Use on previously cleaned surfaces and medical devices. Spray on the areas to be treated until they are totally moistened. Apply in sufficient quantity (+/- 40 ml/m²). Complete wetting of all surfaces to be disinfected is important. Respect the indicated contact time for the required antimicrobial activity. Rinsing is unnecessary except for areas intended to come into contact with the skin or mucosa. For the frequency of use, refer to the hygiene plan in place. Please refer to the manufacturer's instructions for use for cleaning and disinfecting medical devices. Before application, check the compatibility of the areas to be treated with alcohol. Electrical devices being disinfected must be switched off.

INGREDIENTS

Ethanol (CAS N° 64-17-5) 550 mg/g,
N,N-Didecyl-N-methyl-poly(oxyethyl)
ammonium propionate (CAS N°
94667-33-1) 1.1 mg/g, perfume

CERTIFICATES

Medical device and Biocide. Full antimicrobial activity for ANIOSPRAY QUICK is included in the scientific dossier, available on request.

STORAGE

Storage: between +5°C and +25°C

MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.