



# CLEANING AND DISINFECTION SOLUTION FOR MEDICAL DEVICES AND GENERAL SURFACES

## SURFA'SAFE Premium

### DISINFECTANT DETERGENT SOLUTION

Disinfectant detergent solution, intended for cleaning and disinfection of non-invasive and non-submersible medical devices. Biocide: for general surfaces. SURFA'SAFE PREMIUM is a ready to-use surface cleaning and disinfection product.

Combining broad spectrum efficacy, short contact times with excellent material compatibility (including alcohol sensitive surfaces) and the properties of a gentle cleaner.

Its features make SURFA'SAFE PREMIUM suitable for use on a wide range of surfaces in healthcare facilities and an ideal product for intensive care and other high-risk areas.

This enables the products to be used in practice as intended in your protocols, delivering the disinfection results and compliance level mandatory for effective disinfection to restrict the spread of pathogens.

- Good foaming properties.
- Efficient formula without unnecessary ingredients (perfume and dye free).
- Cleaning and disinfection in one step, leaving the surface visibly clean.



**ECOLAB**<sup>®</sup>

[www.ecolab.com](http://www.ecolab.com)

LABORATOIRES  
**ANIOS**  
An Ecolab Company

[www.anios.com](http://www.anios.com)

# SURFA'SAFE Premium

Disinfectant detergent solution, intended for cleaning and disinfection of non-invasive and non-submersible medical devices. Biocide: for general surfaces.

## PRODUCT CHARACTERISTICS

- Cleans and disinfects in one step
- Without alcohol, perfume and dye
- Broad spectrum antimicrobial and optimised contact time
- Compact foam dispenser to limit the formation of aerosols and volatile components (750 ml)
- Formula ensures compatibility with a wide range of materials, including polymers (e.g. polycarbonate, plexiglas)
- Dense foam for better surface adhesion (750 ml)
- pH 6

## MICROBIOLOGICAL PROPERTIES

EFFICACY	CONTACT TIME
Bactericidal - According to: EN 13727* (1 min), EN 16615* (2 min)	2 min
Yeasticidal - According to: EN 13624* (2 min), EN 16615* (2 min)	2 min
Tuberculocidal - According to EN 14348*	60 min
Effective against enveloped viruses according to EN 14476*	1 min
EN 14476* Polyomavirus SV40	5 min
EN 14476* Rotavirus	30 sec
Limited spectrum virucidal according to EN 14476*	30 min
EN 14476* Adenovirus	30 min
EN 14476* Norovirus (MNV)	15 min

\* dirty conditions

## ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
SURFA'SAFE Premium	12 x 750 ml	2419.544



**LABORATOIRES ANIOS**  
1, Rue de l'Espoir  
59260 Lezennes  
France

Distributed by:  
**LABORATOIRES ANIOS**  
1, Rue de l'Espoir  
59260 Lezennes  
France



[www.ecolab.com](http://www.ecolab.com)



[www.anios.com](http://www.anios.com)

## APPLICATION AREA

Disinfectant detergent solution, intended for cleaning and disinfection of non-invasive and non-submersible medical devices.

Biocide: for general surfaces.

## INSTRUCTIONS FOR USE

Ready-to-use solution.

Spray the disinfecting detergent solution on a non-woven cloth and wipe.

Spread evenly the product (+/- 6 ml/m<sup>2</sup> or around 10 sprays). Respect the indicated contact time for the required antimicrobial activity.

Rinsing is unnecessary unless when areas to be treated are intended to come into contact with skin or mucosa.

Please consult manufacturer's instruction for use for cleaning and disinfection of medical devices.

For the frequency of use and cleaning of the application equipment, refer to the hygiene plan in place. Scientific dossier (efficacy, compatibility, etc.) available on request.

## INGREDIENTS

Didecyltrimethylammonium chloride (CAS N° 7173-51-5) 3 mg/g, non-ionic surfactants, chelate, excipients.

## CERTIFICATES


Medical device and Biocide. Full antimicrobial activity for SURFA'SAFE Premium is included in the scientific dossier, available on request.

## STORAGE

Storage: between +5°C and +25°C.

## MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.

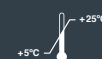

FOR PROFESSIONAL USE ONLY.