

DETERGENT FOR AUTOMATED ENDOSCOPE REPROCESSING

SOLUSCOPE EZ

DETERGENT FOR FLEXIBLE ENDOSCOPES TO BE USED EXCLUSIVELY WITH SOLUSCOPE SERIE 1

- Tri-enzymatic detergent
- Proven efficacy against biofilm and proteins
- Compliant with EN ISO 15883-4
- Compatible with a large range of flexible endoscopes
- Foolproof connection



SOLUSCOPE EZ

Detergent for flexible endoscopes, to be used exclusively with Soluscope Serie 1



SOLUSCOPE SERIE 1

PRODUCT COMPOSITION

- Nonionic and anionic surfactants
- Sequestering agents
- Enzymatic complex (protease, lipase, amylase)
- Excipients

PRODUCT CHARACTERISTICS

- Pale yellow clear liquid
- Density at 20°C: 1.09 mg/ml
- pH of diluted product (0.25%): approx 8.5
- pH pure product at 20°C: about 6.7*
- pH diluted at 0.5%: about 7.6*
- Conductivity at 0.5%: 231 µS/cm

*pH value measurements under operating conditions always by means of a pH meter with temperature compensation.

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
SOLUSCOPE EZ	4 x 5 L	XXXXXX

INSTRUCTIONS FOR USE

Screw the black-capped plunger onto the container. Make sure that the vent hole is not blocked. Put the container in place and connect it to the Washer-Disinfector (refer to Washer-Disinfector user manual). Automated dilution. Shelf life after opening: 2 months while connected to the Washer-Disinfector (write the opening date on the label in the location provided for this purpose).

STORAGE

Dangerous - respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

MEDICAL DEVICE USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.



SOLUSCOPE SAS
(An Ecolab Company)
100, rue du Fauge
Z.I Les Paluds - 13400
Aubagne - France

Distributed by:
ECOLAB EUROPE
Richtstrasse 7
8304 Wallisellen
Switzerland



www.ecolab.com



www.soluscope.com