

DISINFECTANT FOR AUTOMATED ENDOSCOPE REPROCESSING

SOLUSCOPE PA

DISINFECTANT FOR FLEXIBLE ENDOSCOPES

- Peracetic acid-based disinfectant
- Broad spectrum antimicrobial activity
- Compliant with EN ISO 15883-4
- Compatible with a large range of flexible endoscopes
- Lock key system and container with integrated cannula



SOLUSCOPE PA

To be used exclusively with Soluscope Serie 1 PA



SOLUSCOPE SERIE 1 PA

PRODUCT COMPOSITION

- 5% (w/w) Peracetic Acid
- Stabilising agents

PRODUCT CHARACTERISTICS

- Shelf life: 14 months closed
- Aspect: clear and colourless
- Odour: characteristic
- pH pure product*: 1 - 4.2
- Specific gravity at 20°C: 1.10 - 1.13
- Refractive index: 1.35 - 1.37
- Viscosity: < 10 mPa/s

*pH value measurements under operating conditions by means of a pH meter with temperature compensation.

MICROBIOLOGICAL PROPERTIES

CLAIM/CONDITIONS	CONCENTRATION	TEMPERATURE	CONTACT TIME
Bactericidal* - According to EN 13727+A2, EN 14561	1%	40°C	6 mins
Mycobactericidal* - According to EN 14348, EN 14563	1%	40°C	6 mins
Fungicidal* - According to EN 13624, EN 14562	1%	40°C	6 mins
Virucidal* - According to EN 14476+A1, EN 17111	1%	40°C	6 mins

*=clean conditions

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
SOLUSCOPE PA	4 x 5 L	XXXXXX

INSTRUCTIONS FOR USE

- Put the container in place
- Screw the white cap connected to your Soluscope Serie 1 PA onto the container (refer to WD user manual)
- Make sure that the vent hole is not blocked
- Automated dilution: 1% (10mL/L) - 40°C - 6 min

Shelf life after opening: 2 months while connected to the WD (write the opening date on the label in the location provided for this purpose).

STORAGE

Warning - respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

MEDICAL DEVICE USE SAFELY

Class IIb

Please read label and product information.



FOR PROFESSIONAL USE ONLY.